

AactiNose Normal Saline Nasal Drops 10ml

INGREDIENT

Sodium Chloride BP 0.9% w/v

ALSO CONTAINS

Benzalkonium Chloride Solution BP 0.02% v/v

INTENDED USE

Unblock the nose and provide relief from congestion.

HOW TO USE

Adult, Elderly & Children

- Open the nasal drop bottle by breaking the safety seal and tighten the cap to pierce the nozzle.
- Instill one or more drop into each nostril as required or directed by the physician. If symptoms persist please consult physician.

Children Under 5 Years

- Advised to consult medical practitioner or should receive direct medical supervision.

STORAGE INFORMATION

- Do not store before 25°C.
- Do not freeze.
- Keep away from sunlight.

DISPOSAL

Discard the drops 28 days after first opening.

SIDE EFFECTS

Do not use normal saline nasal drops if you have known hypersensitivity to the preservative benzalkonium chloride. The preservative benzalkonium chloride can cause hypersensitive reactions as a side effects. In this case stop using normal saline nasal drops and tell your physician or pharmacist.

IF THE NASAL DROPS GET INTO THE EYES

Take care to avoid the nasal drops getting in your eyes.

If this happens, rinse the eye immediately with water for 10 to 15 minutes, holding the eyelids apart and avoiding the rinse liquid in the other eyes.

Tell your doctor, nurse or pharmacist if irritation persists.

WARNINGS/PRECAUTIONS

FOR EXTERNAL USE ONLY

KEEP OUT OF THE SIGHT AND REACH OF CHILDREN.



	Warning
	Lot No.
	Use By
	Manufacturer
	Please follow the instructions for use
	The product satisfies the General safety and Performance Requirement according to Regulation (EU) 2017/745 of the European Parliament and of the Council of 5th April 2017.
	Conformity with the applicable requirements for product sold with in Great Britain.
	Medical Device
	Not Injectable
	Do not use if the bottle is damaged.
	Store below 25°C.
	Keep away from sunlight



TriOn Pharma



TriOn Pharma Ltd. - UK

Cowplain PO8 8DL, UK

info@trionpharma.co.uk

UK Responsible Person: CMC Medical Devices LTD

Office 32 19-21 Crawford Street, London W1H 1PJ

Email: info@cmcmedicaldevices.com



CMC MEDICAL DEVICES & DRUG S.L.

C/o Horacio Lengo No^o 18, CP 29006, Malaga-Spain

Tel: +34951214054, Email: info@cmcmedicaldevices.com

Date of issue: Jan 2023

Revision: 00

Review date: NA

NSDH00 10ml e