


## Instructions for Use

# *Aactive HA PF* **Sodium Hyaluronate 0.1%**

Soothing & Moisturizing Action

**Preservative Free Eye Drops**

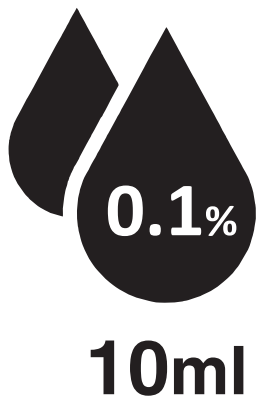

Aactive HA PF is a sterile, soothing and moisturizing ophthalmic solution, with physiological pH and based upon sodium hyaluronate.

Aactive HA PF decreases symptoms of moderate dry eyes such as burning, itching, and foreign body sensation.

Sodium hyaluronate is a viscous polymer used for its moisturizing and lubricating effects. Thanks to its properties it creates a protective, transparent and viscoelastic film on the ocular mucosa, active against external agents and which counteracts moderate dry eyes.

Aactive HA PF acts as humectant and lubricant, providing relief of dry eye conditions, including those associated with the use of contact lenses, PC, infrequent blinking, certain medical treatments, atmospheric pollution, air conditioning, central heating, wind and sun.

Thanks to the OSD closure system which is able to preserve the microbiological, chemical and physical characteristics of the solution. Aactive HA PF does not contain preservative.

The solution is compatible with contact lenses.

### COMPOSITION

Sodium Hyaluronate 0.1%, Sodium Phosphate Monobasic Monohydrate, Sodium Phosphate Dibasic Dodecahydrate, Sodium Chloride, Water for Injection, Preservative free

### INSTRUCTION FOR USE

- Tear off the tamper-evident seal before the first administration;
- Remove the protective cap and open the bottle;
- Instill 1-2 drops in each eye slightly pressing the bottle;
- The gradual emptying of the bottle may be associated with the need to exert a higher pressure to allow the delivery of the product. This is due to the proprieties of the closure system act to preserve the sterility and the chemical and physical characteristics of preservative free solutions;
- After use close immediately the bottle with the cap taking care to shake the bottle once downwards in order to remove any excess product before replacing the cap on the container; do not touch the applicator tip with anything, including your eye or your fingers;
- The content should be re-used within the next 6 months from the first opening;
- It is suitable for frequent use, even several times a day (2 or 3 times or more often if required) or according the doctor indication.

Minor burning/stinging/irritation may temporarily occur as a consequent of the ocular irritation. If any of these effects persist or worsen, tell your doctor promptly.

## WARNINGS AND PRECAUTIONS

- Apply the product before inserting contact lenses
- The product is for single patient use.
- The product is only intended for ophthalmic use.
- Never use the product if the container results, at the time of first opening, damaged or not perfectly sealed.
- During the application, do not touch your eyes and any other surface with the dropper of the container.
- Do not use in the event of known incompatibility with any of the components.
- The administration of ophthalmic drugs concurrently with the eye drops should be carried out under the supervision of your doctor who can evaluate the possible interactions and should be carried out at least 15 minutes after the ocular drops are instilled.
- In case of problems during treatment interrupt the use and consult a doctor.
- Do not use after the expiry date indicated on the package and label.
- Dispose of properly the container after use.
- Keep away from children.
- Do not swallow.
- The product is usually intended for continuous use for no more than 30 days; consult your doctor if symptoms persist.
- Consult your doctor before you use this product if you are pregnant or breast-feeding

## VALIDITY

With un-open pack: Do not exceed the expiry date stated on the box and on the multi dose bottle.

## PACKAGE

10 ml Multi dose bottle, Preservative Free.

## STORAGE CONDITIONS

Do not store above 30°C.

After the first opening the content must be used within 6 months.

**CE** **0546** **STERILE A** MEDICAL DEVICE



Distributed by:

**TriOn Pharma Ltd**

Cowplain PO8 8DL - UK

[info@trionpharma.co.uk](mailto:info@trionpharma.co.uk)

Last Revision 04.02 of 10/2021