

AccuSoft

STERILE EYELID WIPES

- *Preservative Free* ● *Gentle Cleansing*
- *Fragrance Free* ● *Suitable for contact lens wearers*

For Blepharitis, Blepharoconjunctivitis, MGD, Styes, Allergic situation, Pre and post-operative period.

SUITABLE FOR ADULTS AND CHILDREN

AccuSoft sterile disposable wipes are soaked in a natural solution, suitable for use around the eyes, they have iso-lacrima pH, detergent and emollient characteristics. AccuSoft sterile disposable wipes normalize the presence of bacteria on the skin.

PROPERTIES: AccuSoft sterile disposable wipes do not contain fragrances which may cause allergic reactions in sensitive people. It provides gentle cleansing accompanied by a sense of freshness and mechanical removal of mucous secretions and dry or fatty flakes, resulting from inflammatory and/or infectious phenomena, such as blepharitis and blepharoconjunctivitis of various nature, dysfunctions of the meibomian gland, rosacea, styes, chalazion, allergic situations or in case of trauma to the eye area. The product also normalizes the presence of bacteria on the skin and progressively inhibits the reformation of secretions.

AccuSoft sterile disposable wipes can be used by both adults and children from 4 years old of age.

HOW TO USE: Open the sachet by tearing it from the special opening notch. After removing the wipe, possibly with clean hands, proceed to clean the eyelid area, gently passing it over the surface of the eyelid and around the eyes, being careful to keep the eye closed, repeating the step several times to facilitate the complete removal of eye make-up, etc....).

In case of particularly persistent secretions and/or occlusion of the tear duct, it is advisable to heat the sealed sachet for a few minutes in warm water (about 40°C) and then use the towel as a warm compress, massaging gently.

In case of inflammation it is advisable to cool the closed sachet in the refrigerator for a few hours and then apply the open towel on the eye, keeping it for a few minutes as a cold compress.

Since the product must not come into contact with the intraocular area, it can also be used advantageously by contact lens wearers. It can be used as a cleanser of the eyelid area during the pre- and post-operative period in the case of eye surgery. The wipes are sterile and can also come into contact with skin damaged by wounds or abrasions.

COMPOSITION: Vegetable glycerol, Aloe Ferox glycolic extract, Malva Sylvestris glycolic extract, Citrus grandis seed extract hydroglycerin (grapefruit seed extract), Cocamidopropyl Betaine, Monobasic Potassium Phosphate, Dibasic Potassium Phosphate, Disodium EDTA, Purified Water.

WARNINGS: For topical application. Do not apply directly in the eye. Do not ingest. Keep out of the reach of children.

Validity: 36 months.



Rev.00.0324



TriOn Pharma

UKRP & Distributed by:
TriOn Pharma Ltd - UK
Cowplain PO8 8DL
info@trionpharma.co.uk