



Instructions For Use

Sterile Sodium Chloride 0.9% Inhalation Solution

PulmoClear®

FOR INHALATION ONLY — PRESERVATIVE FREE —
DO NOT USE FOR INJECTION

0.9%



20 vials

Isotonic Saline Solution (Sodium Chloride 0.9%)
For respiratory tract humidification

DESCRIPTION

PulmoClear 0.9% is used for moistening the upper and lower respiratory passages by spraying with an electrically operated jet nebuliser or ultrasonic nebuliser; e.g. as a supplementary measure for the treatment of colds (e.g. cough or head cold), inflammation or chronic diseases (e.g. hay fever, asthma, COPD, etc.), reduced humidity typical of spending long periods in air-conditioned or heated rooms. PulmoClear 0.9% restores sufficient moisture to the mucous membranes of your respiratory passages, preventing the discomfort of them drying out. This supports the natural cleaning process of the respiratory tract, a noticeably positive contribution in cases of inflammation as well as in cases of increased exposure to dust and air pollution (e.g. caused by tobacco smoke). PulmoClear 0.9% is not subject to any age limit restrictions and may therefore also be administered to small children.

APPLICATION

Fill the content of a single-dose container into the nebuliser as required, up to a maximum of 4 times a day. Please follow the instructions of the manufacturer of the electrically operated nozzle nebuliser or ultrasonic nebuliser used.

The duration of inhalation depends on the type and equipment of the inhalation device used and may therefore vary slightly.

Preparation of inhalation

Open the nebuliser by gently twisting the nebuliser top anticlockwise, removing it from the container.

Open the ampoule and fill the PulmoClear 0.9% into the container of the nebuliser.

Inhalation with mouthpiece

Take the mouthpiece between your teeth and put your lips around it. Sit upright and relaxed. Breathe in through the mouthpiece and breathe out again through the nose after a few seconds, in a quiet and regular manner.

Inhalation with mask

Hold the mask to your face so as to fit tightly over your mouth and nose. Take Carry out the breathing procedure as described above.

COMPOSITION

Sodium chloride 0.9%, water for injections

PACK SIZES

Single-dose containers, 20 x 2.5 ml

PRECAUTIONS AND WARNINGS

- Do not use for injection!
- Do not use if the single-dose container is damaged.
- Do not use in case of hypersensitivity to any of the ingredients.
- Intended for single use only; as PulmoClear 0.9% does not contain preservatives, do not use leftover solution.
- Do not use the solution after the expiry date. You will find this date stamped on each single dose container and on the folding box.
- PulmoClear 0.9% is in small plastic containers that could be swallowed by small children. Therefore, please keep PulmoClear 0.9% out of the reach of children.
- If you experience a serious adverse reaction during or in connection with the use of this product, please report it to the manufacturer and the competent authority.

STORAGE

Store at 2° C to 25° C. Do not freeze. Protect from light by storing the single-dose containers in the original folding box.

EXPLANATION OF THE ICONS

	Batch code		For one single use only
	Expiry date		Do not use if package is damaged (single-dose container) and consult instructions for use
	Date of manufacture		Medical device
	Sterilized using aseptic processing techniques		Unique device identifier
	Single sterile barrier system with protective packaging outside		Manufacturer
	Consult instructions for use		Distributor
	Temperature limit and storage temperature		Importer
	Keep away from sunlight, store the single-dose container protected from light in the original folding box.		CE marking = symbol of free marketability in the EU with indication of the involved Notified Body