

# Aactive HA PF

## Sodium Hyaluronate 0.1% Preservative Free Unit Dose Eye Drops

20 vials x 0.4ml

### INSTRUCTIONS FOR USE

#### **Applications:**

The Aactive HA PF in the form of eye drops can be used to alleviate symptoms of irritation, dryness and fatigue in the eye. The special formula contains sodium hyaluronate, a naturally occurring substance in the eye, which forms an even, protective layer of film which can absorb large amounts of water and is resistant to evaporation. This helps hydrate the eye, improving visual comfort and making contact lenses easier to wear.

The Aactive HA PF is highly recommended for contact lens users who frequently experience dryness or irritation in the eye.

#### **Directions for use:**

- Wash hands with water and soap prior to using the product.
- Open the bottle of Aactive HA PF by twisting the cap.
- For direct use: Place one or two drops in each eye and blink several times.
- For use with contact lenses: Apply one or two drops to the internal face of the lens prior to placing it on the eye.
- The Aactive HA PF may be used as often as is needed.

#### **Warning:**

- Keep out of reach of children.
- This product does not contain preservatives. Use immediately once opened. Discard any remaining solution after use, as this may become contaminated, which may cause infection.
- Once opened, do not share a bottle with anyone else.
- Do not use product if past its expiration date.
- Do not use if the product appears damaged.
- Do not use the product if you have any allergy to any of its ingredients.
- Should irritation occur, consult your eye specialist.
- Do not store at temperatures above 25°C.

#### **Chemical composition:**

0.1% Hyaluronic Acid, Sodium Chloride, Boric Acid buffer/Sodium Borate with 0.1% Edetate Disodium.



Exclusively Distributed by:  
**TriOn Pharma Ltd. - UK**  
Cowplain PO8 8DL, UK  
info@trionpharma.co.uk



STERILE A



0318